D-Q-SIT AIR

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Table of contents

Preface	8
Declaration of conformity	8
Service life	8
Disposal	8
Identification plate	8
Reuse	8
Warranty	8
Parts of the product	9
Introduction	9
Indication Contraindication	9
Scope of delivery	9
Safety instructions	9
Product description	10
Use	10
Cleaning maintenance	10
Storage	11
Technical data	11

Preface

Thank you for choosing a product from Drive DeVilbiss.

You will not be disappointed by the design, functionality and quality of this product.

Please read these operating instructions carefully prior to initial use. They contain important safety information and valuable tips for the proper use and care. If you have questions or need additional information, contact your Drive DeVilbiss specialist dealer who supplied your product.

These operating instructions contain all of the important information for adjustment and operation. The most current operating instructions are available as a PDF online or can be requested from customer service.

Enlarged formats for visually impaired persons can be displayed. Blind patients must be instructed by their attendant. Repairs as well as some adjustments require special technical training and must therefore be made by the medical supplies dealer.

Declaration of conformity

For the product described, DRIVE MEDICAL GMBH & Co. KG declares compliance according to the medical device regulation (EU) 2017/745.

Service life

Our company assumes a service life of five years for this product as long as the product is used as intended and all maintenance and service requirements are observed.


This service life may be significantly exceeded if the product is carefully handled, repaired, maintained and used and there are no technical limits based on the further development of science and technology.

The service life may also be significantly shortened through extreme and improper use, however. Determination of the service life by our company does not represent any additional guarantee.

Disposal

If you no longer need the product, please contact your specialist dealer or bring the product to your local disposal center.

Identification plate

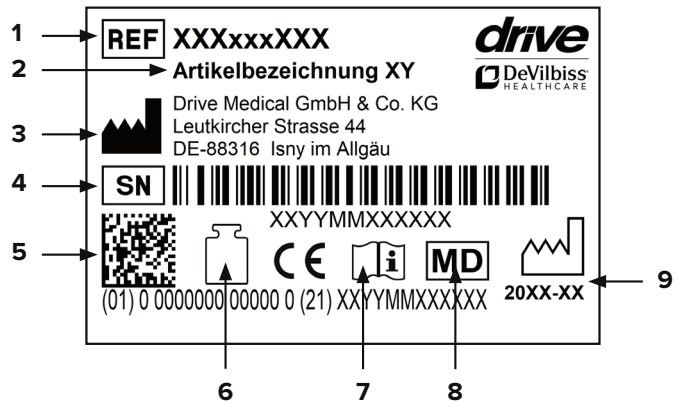1 Article number | 2 Article name | 3 Manufacturer | 4 Serial number | 5 UDI code | 6 Max. load | 7 Follow the instructions for use | 8 Medical device | 9 Production date (year-month)

Reuse

This product can be reused.

Please note that all technical documents needed for safe handling must also be given to the new user.

The product must be checked by the specialist dealer, cleaned, and restored to optimum condition.

Always follow the manufacturer's instructions during preparation; these are made available on request.

Accessories are available from your specialist dealer or at www.drivedevilbiss.de

Warranty

The warranty period for the product supplied by us is 24 months from the date of purchase. Defective parts will be replaced free of charge if it can be shown that there is a defect in materials or manufacturing.

Parts subject to wear are generally not covered during the normal guarantee period unless the parts need repair or replacement which became necessary as a clear, direct consequence of a manufacturing or material error.

We reserve the right of error and modifications on the basis of technical improvements and changes in design.

EXCLUSION OF WARRANTY

Parts subject to wear include (among others):
cover

Parts of the product

D-Q-SiT Air

- 1. cover
- 2. cushion

Introduction

The D-Q-SiT AIR was specially developed for decubitus prophylaxis and to support therapy and is suitable for patients up to 150/200 kg, depending on the model.

It is an air cell cushion with a special D-Q-SiT PROTECT cover for the best possible pressure distribution and shear force reduction while sitting.

The D-Q-SiT AIR is designed for use in wheelchairs.

We expressly ask you to read through the instructions for use carefully so that you obtain the desired medical effect with our product.

Indication | Contraindication

INDICATION:

- Decubitus prophylaxis and support of decubitus therapy up to and including category 4.
- Support for decubitus therapy of categories 1-4 in the case of persons who can regularly change their seated position independently or with assistance.
- For persons who must sit over the long term due to illness or disability.
- To support follow-up treatment for a decubitus which has already healed.

CONTRAINDICATION:

- No known contraindications.

Note

The use of the D-Q-SiT Duo seat cushion as therapeutic support in category 1-4 must always be individually evaluated, assessed and decided on by trained profes-

sionals or the attending physician.

Scope of delivery

Please check the contents for damage and completeness. If one of the parts listed below is missing, please contact your specialist dealer.

1 x	D-Q-SiT Air
1 x	D-Q-SiT protect cover
1 x	Air pump
1 x	Repair kit
1 x	Instructions for use

Safety instructions

- The user (caregiver) must have expert knowledge to be able to estimate the risk of decubitus (e.g. using a recognized scale), assess and evaluate decubital ulcers (decubitus category according to EPUAP) and be able to perform position changes professionally with the patient in a seated position.
- Take note of the permitted maximum weight of the user. This information can be found in the technical data in these instructions for use as well as on the identification plate.
- Protect the D-Q-SiT seat cushion from pointed or sharp-edged objects and surfaces. There is a risk of damage!
- Before each use of the D-Q-SiT seat cushion, ensure that it is in proper and defect-free condition.
- Protect the product from direct sunlight. Product parts can become hot and cause skin injuries if touched.

Product description

D-Q-SIT AIR

Single/dual chamber seat cushion made of individual interconnected air cells. This allows the seat cushion to be individually adapted to nearly every user.

D-Q-SIT PROTECT COVER

Flame-retardant according to BS 7175 Crib 5, waterproof, breathable, can be disinfected by spraying or wiping, certified according to ÖkoTex Standard 100.

Use

Take the seat cushion out of the box.

- Check the wheelchair for potential hazards which can damage the seat cushion; see safety instructions.
- Place the D-Q-SiT Air on the chair/wheelchair such that the fixation straps are in the back so that you can secure the seat cushion to the chair/wheelchair. The air valve must be in the front of the cushion.

IMPORTANT!

Please do not use any other materials, such as disposable sheets, between the surface of the cushion and the user, if possible. This can have a negative effect on the characteristics of the seat cushion.



Adjustments

- Place the D-Q-SiT Air on the chair/wheelchair. The air valve must face forwards.
- Open the valve by turning it to the left (counterclockwise; **photo 1**).
- Connect the silicone hose of the pump to the open valve and pump air into the cushion until it swells slightly.
- Close the valve by turning it to the right (clockwise; **photo 2**).
- Place the user on the D-Q-SiT Air (if this has not been done already) and open the valve again by turning it to the left (counterclockwise). Allow the user to sink down as far as possible into the cushion by allowing air to escape to ensure the best pos-

sible pressure distribution. Once the user has achieved his/her ideal position, close the valve again by turning it to the right (clockwise; **photo 3**).

- The ideal adjustment of the seat cushion is when the distance between the lowest point of the user (generally the ischial tuberosities) and the base plate is approx. 2 cm.
- Check the adjustment as follows: place a flat hand under the ischial tuberosities. If you can press your fingers approx. 2 cm downwards, the D-Q SiT Air is optimally adjusted (**photo 4**).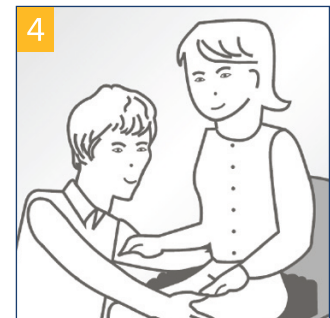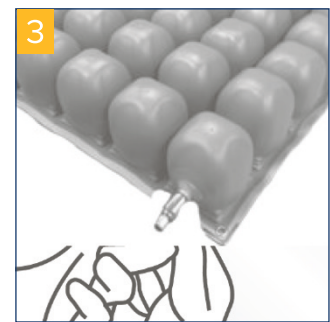

Cleaning | maintenance

Seat cushion core

Can be cleaned by hand. Use a damp (not wet) cloth with warm water and a mild cleaning solution. Do not scrub or brush!

D-Q-SIT PROTECT cover

The cover can be cleaned by hand. To do this, use a damp (not wet) cloth and a mild cleaning and disinfectant solution. You can also wash the cover in the washing machine at max. 60°C using a household detergent. The seat cushion can be disinfected with a spray and wipe disinfection solution such as Bacilol AF. During reprocessing of the antidecubitus seat cushion, take note of the medical device law, the „Guidelines for the Processing of Medical Devices“ as well as the „Guidelines for Hospital Hygiene and Infection Prevention“ of the Robert Koch Institute.

Maintenance

The antidecubitus seat cushion is maintenance-free!

Storage

- Store the D-Q-SiT seat cushion in a clean and dry condition.
- Protect the D-Q-SiT seat cushion from direct sunlight.

Technical data

1 chamber				
Article number	820200310	820200320	820200330	820200340
Overall height	6 cm			
User weight	0 - 150 kg			
Article number	820200410	820200420	820200430	820200440
Overall height	10 cm			
User weight	0 - 200 kg			
Overall depth	40	40	45	45
Overall width	40	45	45	50
2 chambers				
Article number	820200510	820200520	820200530	820200540
Overall height	6 cm			
User weight	0 - 150 kg			
Article number	820200610	820200620	820200630	820200640
Overall height	10 cm			
User weight	0 - 200 kg			
Overall depth	40	40	45	45
Overall width	40	45	45	50
Technical data				
Weight	approx. 1,5 kg			
Aircell material	PVC			
Cover material	Polyester/Nylon, Polyurethane			

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Drive Medical GmbH & Co. KG